FlamiActiv®

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FlamiActiv®

Indications

FlamiActiv® is a hydro-active colloid gel for the treatment of minor wounds which supports the fast healing of skin by covering the wound and creating an optimal moist healing environment. **FlamiActiv®** is used post lasertherapy, on superficial burns (including those caused by radiotherapy) and on minor wounds (e.g. grazes, cuts). **FlamiActiv®** may also be used in superficial open wounds.

Properties

FlamiActiv® hydro-active colloid formulation can quickly help soothe the pain of minor wounds due to its cooling effect. **FlamiActiv®** helps to create optimal healing conditions to accelerate cell renewal, allowing the wound to heal fast and so reduce the likelihood of scarring. **FlamiActiv®** also cushions the wound to provide a barrier against external contamination.

FlamiActiv® can be used for dry or wet wounds:

- In case of a dry wound **FlamiActiv®** turns the dry wound into a moist one ("hydrogel" effect). The dry wound absorbs the water contained in **FlamiActiv®** and promotes the dissolution of necrotic/damaged tissue.
- In case of a wet wound, the hydrocolloid of **FlamiActiv®** is activated and consequently **FlamiActiv®** absorbs excessive fluid from the wound and restores and maintains the water balance in the wound ("hydrocolloid" effect).


How to use

Wash hands before and after treating the wound. Clean and dry the wound gently before applying the gel.

For superficial burns and minor wounds (e.g. grazes, cuts). Carefully apply **FlamiActiv®** to the wound (without rubbing) once or twice per day (in case of sunburn apply twice to six times daily). If necessary, cover the wound with a plaster or dressing and change daily.

For superficial open wounds: Apply a thick layer of **FlamiActiv®** (5 mm) to the wound minimising contact with the wound edges. Cover **FlamiActiv®** with a sterile non-adhesive dressing and fix with gauze or a bandage. Leave **FlamiActiv®** in place as long as the gel structure is intact. When new skin cells start to form and the wound is closing, only apply a thin layer of **FlamiActiv®** to prevent weakening. If necessary, protect the wound edges.

- Open wounds: change daily.

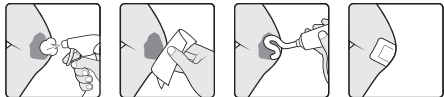clean

dry

apply

cover

Removing the gel:

- Wash hands before and after removing **FlamiActiv®**.
- **FlamiActiv®** can be removed simply and painlessly with wet gauze.

Precautions

- For external use.
- Side-effects such as irritations or allergic reactions are rare.
- Do not use **FlamiActiv®** when there is a known allergic reaction to parabens or any of the ingredients.
- Do not use **FlamiActiv®** on the eyelids or in the eyes. If **FlamiActiv®** comes into contact with the eyes rinse the eyes with running water and consult a doctor.
- On extensive weeping or infected wounds, consult a doctor before using **FlamiActiv®** because other complementary treatments might be necessary.
- If the wound becomes swollen, reddened or painful, or doesn't start to heal within a few days seek medical advice.

Storage conditions

- Store at room temperature (< 25°C), in a dry place and in the original packaging.
- Close the tube after use.
- Do not use **FlamiActiv®** after the expiry date shown on the pack. The expiry date (USE BY: Year/Month) is the last day of the indicated month and year.

Ingredients

Methyl-p-Hydroxybenzoate, Propyl-p-Hydroxybenzoate, Disodium EDTA, Cetearyl polycarbonate (liquid), Cetearyl polycarbonate (solid), Polyacrylate copolymer, L-Arginine, PEG 400, Purified water.

How supplied

Tubes of 10g, 15g, 20g, 40g, 50g, 100g and 250g.
Not all pack sizes may be marketed.

Fabricante/Manufacturer

Flen Health NV,
Blauwesteenstraat 87,
B-2550 Kontich, Bélgica / Belgium
www.flenhealth.com – info@flenhealth.com

FABRICADO EN BELGICA / MADE IN BELGIUM

Date of issue: 2019/07

CE 0344

905-584/B

I156v1