



Tailor made application.

User friendly wound treatment.

How to use Flaminal® to fill a cavity wound.

How wet is the wound?

Slight to moderate exudate



Cleanse the wound if necessary

Moderate to high exudate



Cleanse the wound if necessary

Fill the cavity with Flaminal®



Using a nozzle



Using a syringe



Direct from the tube

Apply secondary dressing depending on the levels of exudate

PLEASE NOTE:

- ▶ Flaminal® (Hydro or Forte) can remain in place as long as the gel structure is intact (1 to 4 days, depending on the amount of exudate).
- ▶ Once opened, if recapped carefully, a tube of Flaminal® (Hydro or Forte) can be stored at room temperature and used for 12 months.
- ▶ Only 1 primary dressing needed when using Flaminal®.
- ▶ The FIRST time a patient is treated for wounds which might reach the level of bones and joints or with exposed bones and joints, the patient should be under observation for at least 30 minutes after the administration of Flaminal®.
- ▶ Flaminal® can be used in cavities up to any depth that can be probed- as in a case of allergic reaction it must be able to be removed at any given time.

For more information visit flenhealth.co.uk or email info@flenhealth.com

Reference: Flaminal® Instructions for use 10/2021