

Mask related superficial skin injuries


Self-care guide

You have been advised to use Flamigel® by your HCP to treat your superficial mask-related skin injuries. Please follow these guidelines and read the instructions for use carefully.

How to apply Flamigel®



Wash and dry your hands before use.

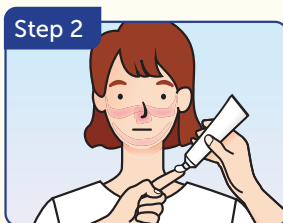

Carefully apply a thin layer of Flamigel® to the affected area of skin. Do not rub in.



Re-apply your face mask when Flamigel® is absorbed.

Flamigel is safe for regular and continuous use

Benefits:

Flamigel® is a hydro-active colloid gel for the treatment of minor wounds which supports fast healing of skin by creating an optimal moist healing environment.

- Cools and soothes the skin
- Non-sticky and non-greasy
- Heals wounds up to **40% faster**
- Creates a **protective skin barrier**
- Fragrance free

Contraindications:

Do not use if you have a known allergy to one of the components.
Do not apply to eyelids or in the eye.
Should it come into contact with your eye, rinse the eye thoroughly.

Flamigel®

www.flenhealth.com



Flamigel® is available in a tube of 50g

If you have any questions relating to the use of Flamigel® please contact medical@flenhealth.com. Please also find further information at www.flenhealth.com. If you have any questions relating to your superficial skin injuries then we advise that you contact your nurse, doctor or pharmacist.

CE 0344